

INSTRUCTION SHEET FOR DRI-AID

The enclosed Dri-Aid container adsorbs moisture from the air. Place Hearing Aid together with the Dri-Aid inner container inside the pouch provided. Roll down the top and snap shut. When ready to use the Hearing Aid unfold the pouch, remove the Aid only and re-seal the pouch with the metal canister inside.

When the chemical in the window turns pink, heat container in an oven at approximately 350°F until blue color is restored. Allow to cool before re-using - repeat as required.



HAL-HEN Garden City Park, NY 11040