


SPrint™ Battery Pack

for two batteries



To replace the batteries,

1. Press the front arrow down and slide the cover away from the controls.

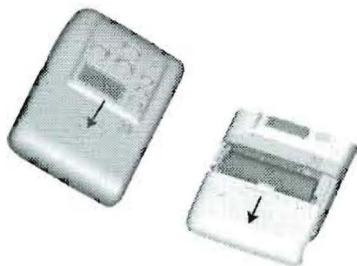


2. To release the battery, press down on its negative end. Remove and dispose of them as required by your local authorities.



Press to release

3. Replace the batteries, taking care they face the correct directions, as indicated in the battery case.

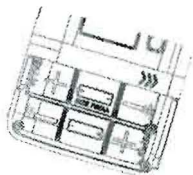

4. Place the cover on the battery case.



5. Slide the cover towards the controls until it clips into place.



To remove the SPrint™ battery case:



Push button down
to release

Slide battery case
to remove



Cochlear Ltd (ABN 96 002 618 073) 14 Mars Road, Lane Cove NSW 2066, Australia Tel: 61 2 9428 6555 Fax: 61 2 9428 6352
Cochlear Americas 400 Inverness Parkway, Suite 400, Englewood CO 80112, USA Tel: 1 303 790 9010 Fax: 1 303 792 9025
Cochlear Europe Ltd 22-24 Worpole Road, Wimbledon, London SW19 4DD, United Kingdom Tel: 44 20 8879 4900 Fax: 44 20 8946 9066
Cochlear AG Margarethenstrasse 47, CH - 4053 Basel, Switzerland Tel: 41 61 205 0404 Fax: 41 61 205 0405
Cochlear GmbH Karl-Wiechert-Allee 76A, D-30625 Hannover, Germany Tel: 49 511 542 770 Fax: 49 511 542 7770
Nihon Cochlear Co Ltd Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan Tel: 81 3 3817 0241 Fax: 81 3 3817 0245
Cochlear (HK) Ltd 21F Shun Ho Tower, 24-30 Ice House Street, Central, Hong Kong Tel: 852 2530 5773 Fax: 852 2530 5183

www.cochlear.com

Cochlear and SPrint are trademarks of Cochlear Limited.

Printed in Australia
N30523F ISS1