

HOW TO USE SOUND-TIPS:

Small screw tips (gold and silver) are sound-tips.

They are used to adjust the tone quality in the following ways:

- a) When a gold sound-tip is inserted in the earpiece, low pitched tones become more audible.
- b) When a silver sound-tip is inserted, medium-pitched tones become more audible.
- c) Without a sound-tip, high, medium and low sounds can be heard equally.

As these sound-tips respectively produce different qualities, it is advisable to select the tip by which you can hear most efficiently.

The way to insert the sound-tip in the earpiece is illustrated on the other side.

The sound tip
must be released
or inserted as
illustrated.



Screw the sound-tip clockwise,
to be inserted.

Screw it counter-clockwise to be
released.

THE M. CALIG COMPANY

355 South Broadway (Suite 507)
Los Angeles 13, California, U.S.A.

MAdison 5-2509

Manufacturers of Calitone Electronic Products.