

In Case Of Difficulty

1. Try a fresh battery.
2. Clean battery contacts with a rubber eraser.
3. Check cord for possible crack or breakage which could cause short circuit. Such breakage is most likely to occur at either end rather than in the middle.
4. If trouble persists, see your *Vicon* dispenser.

How to operate and care for your Vicon Model M-8 hearing instrument.

Your new *Vicon* is a complex and delicate electronic instrument. It should be treated with respect.

This instrument should not be subjected to severe shock. Both the microphone and receiver may be damaged by dropping. Immersion of this instrument in water must be avoided.

Please follow the operation instructions carefully.


Vicon Model M-8

ON-OFF SWITCH




As shown in the diagram at left, the ON-OFF-TEL switch is positioned on one corner of the instrument. To turn the instrument ON, turn the switch lever to the ON position.

VOLUME CONTROL

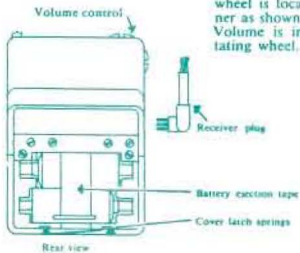


The volume control wheel is located on the corner as shown in the diagram. Volume is increased by rotating wheel.

HOW TO INSTALL AND CHANGE BATTERIES

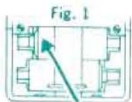

DUMMY
BATTERY

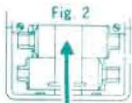

BATTERY
EJECTION
TAPE

Open battery compartment by lifting gently at the bottom of the instrument. It opens very easily and requires very little pressure.

If using one battery, place live battery in as shown and dummy in as shown. Place the + sign so that it corresponds with the + in the bottom of the battery compartment. When replacing battery, also remove dummy cell and clean dummy contacts.

If using two batteries, place in battery compartment as shown in Figure 2. Your dispenser will determine whether you require one or two battery operation for maximum utility. **DO NOT CHANGE FROM ONE TO TWO CELL OPERATION OR VICE VERSA.** This change must be made by your dispenser because internal changes are required.

USING YOUR M-8 ON THE TELEPHONE



For best results when using the telephone, move the switch lever to TEL. Place receiver next to your M-8 as shown. You will soon learn the proper placement of the telephone receiver next to your M-8 for the best telephone reception.

The telephone switch is designed to give you clear, undisturbed reception by cutting the microphone off completely while in the TEL position so that background noises are eliminated.

For Best Results

For best results and greater comfort, keep your ear mold clean. If you are wearing a head borne instrument, make certain that the tubing is free of kinks. Your Vicon Dispenser has ear mold cleaning material available. See him for further instructions.