

Your Telex guarantee entitles you to Telex repairs and service, excepting batteries, for one full year. If you have difficulty with your Telex that you cannot remedy yourself, contact your nearest Telex dealer.



hearing
with your

TELEX



Telex Laboratories

the name **TELEX**
is a U. S. Reg. Trademark.

Where the world's finest precision hearing aids
are manufactured, Telex Park, St. Paul 1, Minnesota.

A special message to you:

You are now the owner of a Telex, the finest small hearing aid in the world. The results of years of research by hundreds of people, the Telex hearing glasses give you more comfortable, more natural hearing—with the same pure, clear tone that has made Telex instruments famous around the world. Worn on the head, it eliminates forever the clothing noise, dangling cords and the bulky inconvenience of conventional hearing aids.

By carefully following the operating instructions in this booklet, you will enjoy the fullest hearing benefits as well as the greatest possible economy with your new Telex hearing glasses. If you need further assistance, rely upon your Telex dealer to help you. See him, too, for fresh batteries and accessories. We are all at your service to give you the clearest and most comfortable hearing possible.

President

how to operate your **TELEX**



- 1 Remove the battery slide. Place the battery in the slide. Replace the slide in the instrument so that the side of the battery marked + is facing outward from the head.



- 2 Place the instrument and glasses on the head with the instrument behind the ear.



- 3 Turn the coupling of the tubing so that the tubing lies securely over the top of the ear.



- 4 Insert the ear tip into the ear canal gently but securely.



- 5 The small lever protruding from the back of the instrument is the Volume Control.

If the instrument is worn on the left side: the uppermost position of the lever will give you the least volume. You get the most volume when the lever is pushed to the maximum downward position.

If worn on the right side of the head: the downward position of the lever will give you the least volume. You get the most volume when the lever is pushed to the maximum upward position.

The Volume Control lever does *not* turn the instrument off or on. Instrument is off when battery is removed, goes on when battery is again inserted.

about the **battery**



Your Telex requires only one power capsule, type RM-400. This capsule will last approximately 25 hours under normal conditions and usage.

IMPORTANT: Use only type RM-400 or other capsules recommended by your Telex dealer. *Always remove capsule when instrument is not in use.*

for the most hearing pleasure from your **TELEX**

If you have been hard of hearing for an extended period of time, the first sounds you hear may seem strange. Be patient—give yourself time to relearn sounds you may have forgotten. It often takes a new user as long as two months or more to become thoroughly familiar with a hearing aid.

Relax. Don't strain to hear. Simply concentrate on what you want to hear and forget you are wearing a hearing aid.

Listen to only one person at a time. Watch the person talking, just as though you had normal hearing. Later you will easily understand conversations of several people. Before attempting group conversations, learn which volume level is best for you.

Listen to music on television or radio. For the first few weeks avoid exciting drama because the sound levels change so quickly.

Take part in social activities after the first two weeks. Never be embarrassed to ask people to repeat what they have said. Repetition is the key to learning any skill. Notice how often people with normal hearing ask to have things repeated.

Audience listening is always the most difficult. Do not attempt it until you have mastered the preceding steps. At church, the

theatre, or any large auditorium, try to sit near the front, in the center or near a wall or pillar. Do not sit under a balcony.

Use your Telex constantly after you have become accustomed to it. By wearing your Telex hearing glasses all day, you become normally adjusted to the world of sound, of which noise and unexpected sounds are a necessary and interesting part.

how to care for your

TELEX

- Give your Telex the same care and consideration you would give a fine watch or any precision instrument.
- ☞ Never expose your Telex to extreme heat, such as in a tightly closed car in summer, or close to a

radiator in winter. Heat, moisture and high humidity are enemies of all things electrical.

- Clean your ear tip daily. The simplest, most effective way to keep your ear tip sparkling clean is to leave it overnight in Tele-Kleen, a special solution that completely dissolves all accumulated wax. Tele-Kleen is prepared by our own Telex Earmold Laboratories. Be sure to clean ear tip carefully with Tele-Kleen and tissue or cotton tipped swab before permitting others to try your Telex. A pipe cleaner may be used to clean opening of ear tip.

Telex hearing glasses have been "torture tested" in heat, moisture, under severe vibration conditions, and in actual wear by users to assure you that it is rugged, dependable, and will give complete hearing satisfaction. If, by chance, your Telex should not operate properly (perhaps due to mishandling in transportation), check the following points before you call your Telex dealer:

1. Are you sure your Telex is on? Check the battery slide and the volume control.
2. Have you checked your battery? Always carry a spare battery with you.
3. Is there wax in your ear tip? If so, the sound cannot get through. Remove it from the instrument and clean the tip with Tele-Kleen.
4. Is there wax in your ear canal? Occasionally check your ear for normal accumulation of wax. Visit your doctor periodically for an ear examination.