

INSTRUCTIONS FOR ACOUSTICON HUMIDRIER

This plastic container together with the bag of moisture absorbing crystals will prevent your hearing aid from losing power due to moisture absorbed during the day.

Place your hearing aid in the plastic bag upon retiring. Moisture absorbed by the hearing aid during the day will then be removed and transferred to the crystals. At the same time the instrument will be protected against additional moisture during the time the hearing aid is not in use.

Keep the plastic bag tightly closed at all times to prevent the crystals from absorbing unnecessary moisture.

Once every thirty days—or more often during the humid summer months—remove the small bag of crystals from the plastic container and dry the crystals in a moderately hot oven for one hour.

Be sure to keep the temperature low enough to prevent scorching of the bag.

DICTOGRAPH PRODUCTS, Inc.
Jamaica, New York